



Pfizer BNT162b Vaccine

- 2ml vial (dimensions 13mm diameter and 31mm height)
- 5 doses per vial. Dose is 0.3ml IM.
- Preparation and administration protocol included in the Pfizer Information for Healthcare Professionals document:

https://www.gov.uk/government/publications/regulatory-approval-of-pfizer-biontech-vaccine-for-covid-19







0.9% NaCL Sol for Injection (BNT162b)

- Diluent for BNT162b (require
 1.8ml per vial).
- Use a fresh amp for each vaccine vial.
- Sourced from two suppliers,
 Fresenius and Bbraun. Both
 pack sizes are 20 x 5ml plastic amps.





CNS for Dilution (BNT162b)

- Needle and syringe for dilution (not used for administration to patients)
- Combined needle and syringe (CNS) sourced from two suppliers, BD (Discardit) and Unisurge.
- Same specification: both 21G x 38mm (1.5 inch) needle with 2ml Syringe with graduations at 0.1ml. 100 pack.
- Needle needs to be attached to the syringe
- Require one per vial.





CNS for Administration (BNT162b)

- BNT162b is 0.3ml IM dose. GBUK (Prosum) product sourced. 1ml syringe with 0.1ml graduations.
- Two sizes, 23g and 25g (1 inch).
 Plan to use 23g until exhausted and then switch to 25g.
- Luer slip fitting (can switch needle).
- No safety feature.
- One per patient. The same syringe will be used for drawing up the vaccine and administration to the patient (no latex in vial topper).



AZ Vaccine (Packaging)

• Dose is provisionally 0.5ml IM (subject to change). 10 vials in a pack.

National Services Scotland

- Pack size depends on the packaging line:
 - 80 dose pack: Wockhardt (UK): 5ml vial - 8 doses per vial. (88% of supply is this pack size)
 - 100 dose pack: IDT (Germany): 6ml
 vial 10 doses per vial.
- Not authorised for use yet. No dilution step.
 Same needle used to draw up and administer (no latex in vial topper).



CNS for Administration (AZ)



- Combined needle and syringe (CNS) sourced at UK level.
- 0.5ml IM dose. BD product sourced is
 0.25ml graduations.
- Two sizes, 23g and 25g. Plan to use 23g until exhausted and then switch to 25g.
- BD is fixed needle (therefore can't switch needle).
- No safety feature.

